

ReflexPillow™

Reflexpillow™ is a patented medical device designed by a specialist to provide several health benefits along with comfort and support.

HOW TO USE

Place the flat side on the bed

Reflexpillow™ has a left and a right side with a ridge in the middle, when you lie on the right side you face to the right side, when you lie on your left side you face to the left side, you do not lie on the middle of the pillow.



PILLOW CARE

- Take care when removing and fitting pillow cover by supporting zipped corners.
- Zip-on case should be removed and washed separately from the body of the pillow.
- Spot-clean inner cover with a cloth dipped in a mild sudsy solution; rinse with a damp cloth.
- Allow the pillow to air dry completely before putting it back on the bed.
- Do not immerse pillow in water!

RECOMMENDATIONS

- Please allow three nights to adapt to your reflexpillow.
- Use loose pillow case for maximum benefit.
- We recommend you change your reflexpillow after three years.

*Please retain instructions for further reference.

Single User only. This pillow is not recommended for children.

Reflex Pillow stored at room temperature before use.

Don't place heavy weight on pillow during storage.

Bag: Risk of suffocation.

Please retain instructions for further reference.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the Competent Authority of the Member State in which the user is established.



GTIN (01): 05391531740424

LOT (10): 0515380

Manufacture Date (11): 2021-01-07



ReflexPillow Ltd.,
Dutec, B5/6 Corcanree Business Park, Dock Road,
Limerick V94 VYX9, Ireland
Phone: +353 61 225677 | info@reflexpillow.com

Wash as synthetics. Tumble dry low heat. Above instructions are for cover not pillow.